

INTELECT MOBILE 2 SOFTWARE UPGRADE INSTRUCTION

PURPOSE

The purpose of this document is to provide a detailed procedure to do a software upgrade of an Intellect Mobile 2 device. This procedure applies to both Windows and Mac OS users.








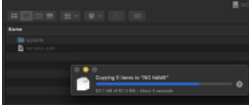
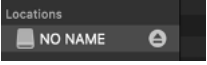
Screen captures below have been taken with an upgrade to version 02.10.06, but this procedure is applicable to any software upgrade.

Important: if you are upgrading your device to version 02.10.06 then apply also the procedure "IntellectMobile2_BluetoothFix" once the upgrade is done.

PROCEDURE


1. Erase the USB drive supplied with the Intellect Mobile 2.
2. Go to the Chattanooga website www.chattanoogarehab.com
3. Go to Intellect Mobile 2 product page
4. Complete the registration form to be informed about new product software version availability and IFU updates (if not already done before)
5. Go to the Software Downloads tab
6. Download firmware upgrade zip file and extract the file. See below separate download instruction for Windows[®] and MAC[®]
7. Switch OFF the device
8. Insert USB key drive into the USB port on the back of the device

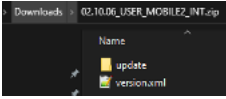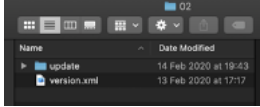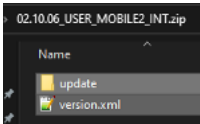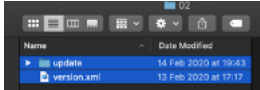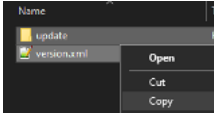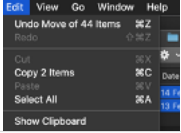

MICROSOFT WINDOWS	APPLE MACOS
1. Open the downloaded zip file, its content will look like the following:	1. Open the downloaded zip file, its content will look like the following:
	
2. Select both Update folder and version.xml	2. Edit → Select all
	
3. Copy the 2 items	3. Edit → Copy 2 items
	
4. Open the USB drive	4. Open the USB drive location
5. Paste the 2 items on the USB drive	5. Edit → Paste
6. Wait for the copy to be over	6. Wait for the copy to be over
	
7. Eject the USB drive	7. Eject the USB drive
	


9. Switch ON the device
10. Device will automatically detect firmware update availability and commence upgrade; the upgrade will take some minutes with an animation on knob LEDs Important: the power must not be switched off during the upgrade
11. Once firmware update is finished, Home screen will be displayed, and the USB drive can be removed. Device is ready for use.
12. Check software version in settings:

a. Open the settings menu

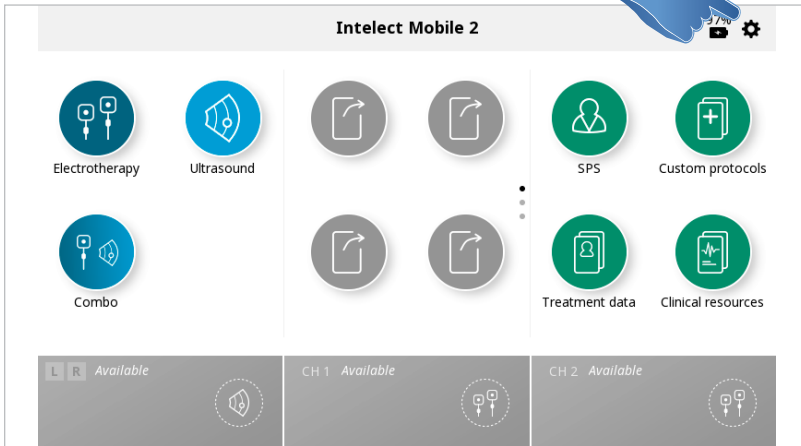

b. Select "Display Unit Version Information"

c. Check the software version is 02.10.06

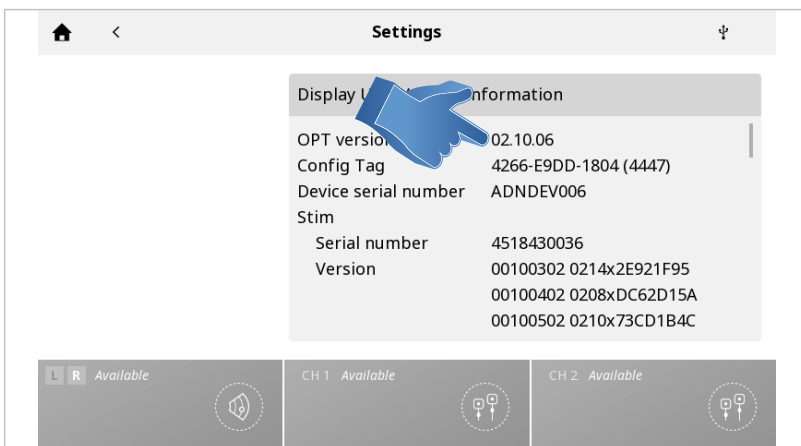

DJO France SAS
 Centre Européen de Frêt 3 rue de Bethar
 64990 Mouguerre · France
 T: + 33 (0) 5 59 52 86 90 · F: + 33 (0) 5 59 52 86 91
 DJOglobal.com



DJO, LLC | 1430 Decision Street | Vista | CA 92081-8553 | U.S.A.

www.chattanooga rehab.com