

## NEO SOFTWARE UPGRADE INSTRUCTIONS


1. Insert the USB flash-drive, provided by DJO, into the USB connector on the device.



2. Power on Unit with flash-drive installed in USB port. Allow initialization to complete.
3. Enter utilities screen by pressing the “Utilities” button on the home screen.



4. On the utilities screen, press the “Display Unit Version Information” screen.

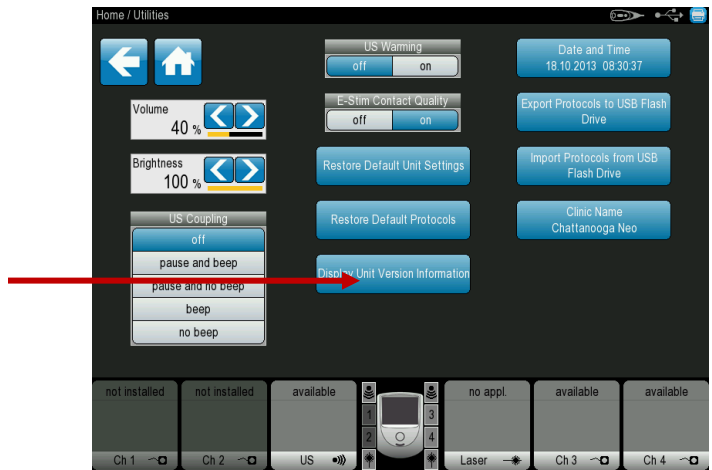

5. Press the “Upgrade Unit Software from USB” button to apply upgraded software. The Screen displays status of the upgrades being installed and the progress of the installation.



**NOTE: DISREGARD ALL PREVIOUS VERSIONS OF USB FLASH DRIVES AND CD’S. RETAIN ONLY THE LATEST REVISIONS INCLUDED IN THIS KIT.**



DJO France SAS  
 Centre Européen de Fret  
 3 rue de Bethar  
 64990 Mouguerre, France  
 T: + 33 (0) 5 59 52 86 90  
 F: + 33 (0) 5 59 52 86 91  
[http://www.djoglobal.eu/fr\\_FR/index.html](http://www.djoglobal.eu/fr_FR/index.html)



© 2017 DJO, LLC. All rights reserved.